



Eagle Eye Anesthesia

11233 St John's Industrial Parkway S.
Jacksonville, FL 32246
800-760-6976
www.eagleeyeanesthesia.com

Installing the Double E-tank Yoke Assembly on your Millennium or other pole mounted machines:

Using the enclosed 5/16th inch Allen wrench remove the two bolts that attach the two main blocks of the e-tank manifold and separate the two halves (*Figs 1 and 2*). Position the main blocks in place on the pole using the heights / locations recommended below. Make sure that the pressure gauge on the main blocks have the face of the pressure dial facing up (*Fig 3*). Install the two bolts and tighten so that the two pieces of the bracket close tightly on the pole (*Fig 4 and 5*). (Tighten the bolts as far as they will go.) Install the oxygen tanks onto the yokes of the manifold making sure that there is enough clearance to freely place the tank into and out of the yoke as well as the operation of the tank valve.



(Fig 1)



(Fig 2)



(Fig 3)



(Fig 4)



(Fig 5)



Eagle Eye Anesthesia

11233 St John's Industrial Parkway S.
Jacksonville, FL 32246
800-760-6976
www.eagleeyeanesthesia.com

Recommended Mounting Heights:

(Standard oxygen e-tank height: 28.5")

Mounting of e-tank assembly for standard e-tank, the measurements to be taken along the mounting pole from the top of the base up towards the machine:

Recommended:

For tank to be situated ABOVE base legs:

Height from the top of the base to top of assembly bracket: 27"

Optional:

For tank to be between base legs:

Height from the top of the base to top of assembly bracket: 24"

Please contact us at 800-760-6976 if further assistance is needed.