





Eagle Eye Anesthesia

11233 St John's Industrial Parkway S.
Jacksonville, FL 32246
800-760-6976
www.eagleeyeanesthesia.com

Top Tray and Bag Hanger Arm/IV Pole Combo Installation for Generation 2 Millennium

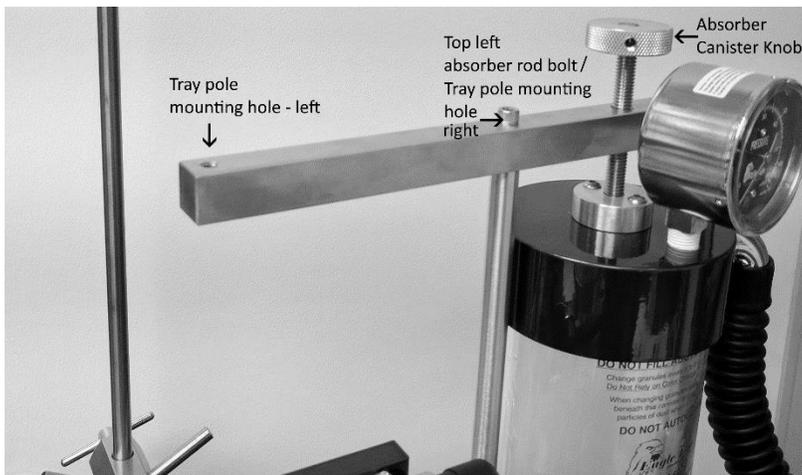

Loosen the **Absorber Canister Knob** and remove the soda sorb canister for this procedure. Using the enclosed 3/16th Allen wrench remove the **Top Left Absorber Rod Bolt**. **This bolt can be discarded.** This will reveal the **Tray Pole Mounting Hole-right**.

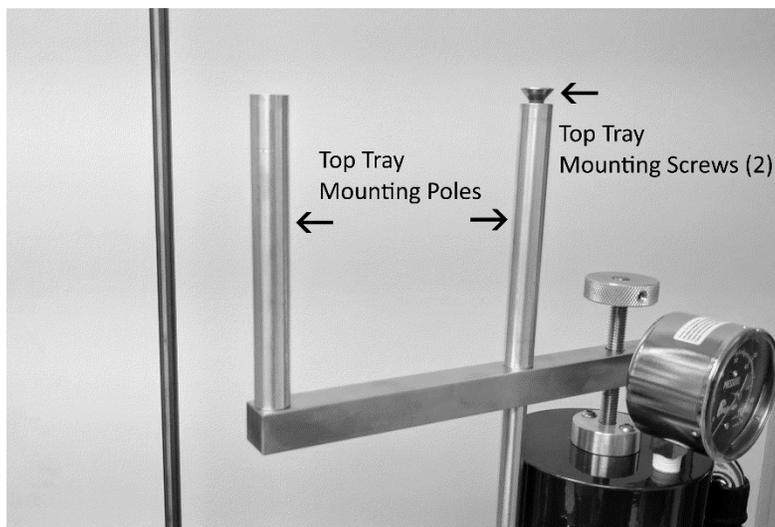

Place both Tray mounting poles into the mounting holes (right and left) and tighten completely (best if done by hand to avoid scratching the brushed aluminum surface). There should now be two mounting poles (Left and Right), one top **right** absorber rod bolt (never removed) in place.





Eagle Eye Anesthesia

11233 St John's Industrial Parkway S.
Jacksonville, FL 32246
800-760-6976
www.eagleeyeanesthesia.com



The **Bag Hanger Arm and IV Pole Combo** is mounted on the back of the vaporizer mounting plate. The **Mounting Screws** are installed from the front of the vaporizer plate, through the plate and into the **Bag Hanger Arm**.

Remove protective film from the top surface of the tray. Once the poles are tightened, place the stainless-steel tray on top of the mounting poles, align the holes and secure the tray by installing and tightening the **2 Top Tray Mounting Screws**. The absorber canister can now be replaced and the absorber canister knob retightened.

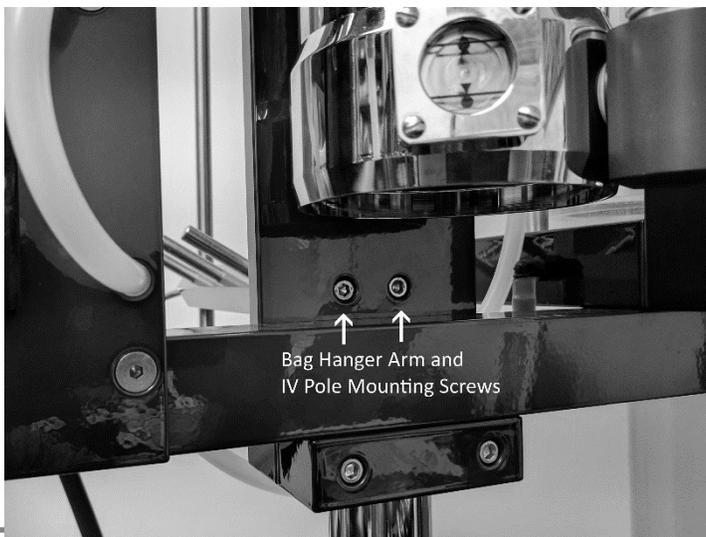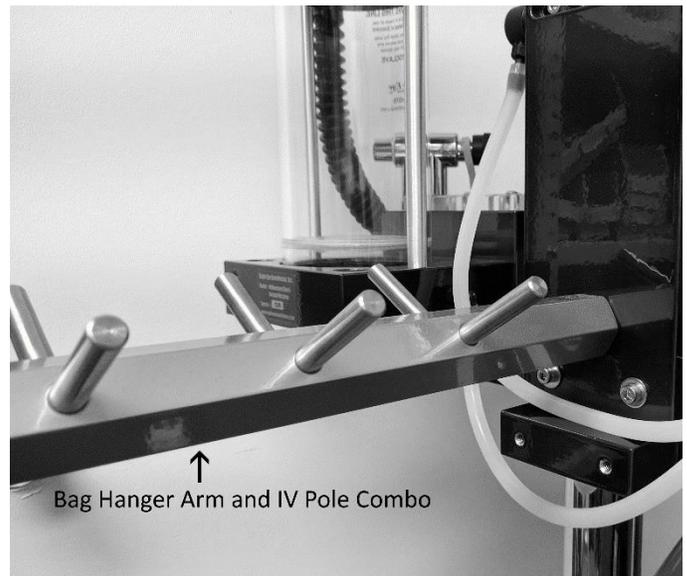

PRO TIP: Before tightening the Bag Hanger Arm completely, install the IV Pole and extend as tall as possible. Using the IV Pole as a vertical guide, hold the Bag Hanger arm straight, keep the IV Pole as vertical as possible, and tighten the Bag Arm Mounting Screws. Take a step back and admire your handiwork. If the IV Pole is leaning to the left or right, loosen one of the two Mounting screws, rotate the Bag Hanger Arm until the IV Pole is no longer leaning, and retighten the Mounting Screws.